# procare

## Compact Compressor Nebulizer

Model# : 240392

**Please read lines instruction manual completely before operating this unit.**

**Caution** - Federal (U.S.A.) restricts this device to safe, by or on the order of a physician

### Components

01. Compressor housing
02. Lid
03. Air vents
04. ON/OFF switch
05. Power cord
06. Air outlet
07. Filter holder

### Standard accessories

08. Nebulizer
09. Mouthpiece with T-connector
10. Tubing
11. Adult mask
12. Pediatric mask
13. Air filter (5pcs)

### Instructions for use Nebulizer

This device is an AC-powered air nebulizer compressor system intended to provide a source of compressed air for medical purposes for use in home healthcare environment. It is to be used with the nebulizer kit to produce aerosol particles of medication for respiratory therapy for both children and adults.

### Important:

Read these instructions carefully before using this device.

### General information

The compressor nebulizer system is for aerosol therapy and it is supplied with the nebulizer kit.

### Caution

- 1. Use the device only for its intended purpose as described in these instructions for use as a compressor nebulizer system for aerosol therapy. Follow the instructions of your doctor.** Any other form of use constitutes improper use and may be dangerous. The manufacturer cannot be held liable for any damage caused by improper, incorrect or unreasonable use, or if the unit is connected to electrical installations not complying with safe regulations.
2. Keep this manual for future reference.
3. Do not operate the unit in the presence of any anesthetic, inflammable mixtures or oxygen.

4. Device operation can be affected by electromagnetic interference that exceeds the limits indicated by the USFDA standards in force. If this device interferes with another electrical apparatus, remove it and plug it into a different electrical outlet.
5. In case of failure and / or malfunction, do not open the compressor housing as this is not a serviceable product.
6. The proper function of the device can be compromised if original replacement parts (e.g. filter) are not used.
7. When using any electrical appliance, certain important safety measures must always be observed, including the following:
  - Use only original accessories and components
  - Never submerge the unit in water as the device is not protected against water penetration
  - Never touch the unit when your hands are wet or moist
  - Do not leave the unit outdoors
  - Place on stable and horizontal surface while operating
  - Make sure the air vent openings are not obstructed
  - Do not allow children or physically challenged individuals to use the unit without supervision
  - Do not turn the unit OFF by simply pulling out the plug form electrical outlet
8. Make sure that the electrical rating shown on the rating plate at the bottom of the unit corresponds to your main voltage and frequency before plugging in the device.
9. Do not use any adapter (simple or multiple), and / or extension cable.
10. Unplug the device from the electrical outlet when not in use.
11. The power supply cord should always be fully unwound in order to prevent overheating.
12. Before performing any cleaning operation or filter replacement, disconnect the device from the main supply by unplugging it or by turning the main supply OFF.
13. Some parts of the unit are small enough to be swallowed by children. Do not let children or physically challenged individuals have access to the unit without supervision.
14. If you decide not to use the device, it is recommended to dispose of the device according to the local regulations in force.
15. Remember to:
  - Use this device only with medication prescribed by your doctor

- Use the accessories recommended by your doctor for therapy

16. Verify that the medication prescribed by your doctor can be used with common compressor nebulizer systems for aerosol therapy.

### Instructions for use

Examine the unit before the first use in order to check for possible damage.

1. When taking treatment with the device sit upright and relaxed.
  2. After unpacking the device, check the compressor and accessories for visible damage or defects such like cracks in the plastic housing which could expose electrical components.
  3. **Before using the device**, proceed with the cleaning operations as described in “**Cleaning and disinfection of the accessories**” section
  4. Open the nebulizer by turning the top counter clockwise.
  5. Make sure the baffle is positioned correctly inside the nebulizer chamber.
  6. Put the prescribed quantity of medication into the medicine cup. Maximum fill volume is 6 ml.
  7. Close the nebulizer by holding the bottom and turning the top clockwise making sure that the unit is properly closed.
  8. Connect one end of the air tube to the nebulizer and the other end to the air outlet on the device.
  9. Connect the mouthpiece to the nebulizer via the T-connector.
  10. If using the mask, remove the T-connector & mouthpiece and connect the mask directly to the tubing.
  11. Plug the compressor into the electrical outlet making sure the electrical rating shown on the rating plate on the bottom of the unit corresponds to your main voltage.
  12. To start the treatment, switch the device on by pressing the ON/OFF by setting the switch position to “I”.
- ATTENTION:** This device is rated for intermittent use with 30 minutes ON/ 30 minutes OFF.
13. When the treatment has been completed, switch the unit OFF by setting the switch position to “O” and disconnect the plug from the electrical outlet.
  14. Wash the nebulizer and accessories as described in

the “Cleaning and disinfection of the accessories” section.

### Cleaning and maintenance

#### Cleaning of the compressor

Use a clean dry cloth and mild detergents to clean the outside of the unit.

**ATTENTION: During cleaning, make sure the internal parts of the unit do not come in contact with liquids and the power cord is not connected.**

#### Cleaning and disinfection of the accessories after each treatment:

The nebulizer can be cleaned by washing in liquid soap and water and then disinfected by boiling in water (max. 10 minutes) or in an autoclave (max. 121°C for 10 minutes). It is suggested that the mask, T-connector and mouthpiece be disinfected using cold disinfecting liquids. Dry the nebulizer using a clean dry cloth. After drying, connect it to the air outlet and switch the compressor ON to verify that it is functioning after cleaning.

**Do not boil or autoclave the T-connector, mouthpiece, tubing or masks**  
**It is important that the cleaning instructions are carefully followed to ensure optimal operation.**

#### Replacement of the nebulizer.

The nebulizer should be replaced after a long period of inactivity, or if it is obstructed by dry medication, dust etc. With normal use, your nebulizer should last at least 6 months.

#### Replacement of the filter

The filter is to be replaced after approximately 30 hours of use, or when it appears dirty. Open the filter holder to replace the filter with a new one.  
Do not use the device without the filter.

#### Troubleshooting

##### The device does not turn on:

1. Make sure the plug is firmly inserted into the electrical outlet.
2. Make sure the device has been operating within operating limits (30 minutes ON/ 30 minutes OFF)

**The device does not nebulize, or the nebulization is weak:**

1. Make sure the ends of the air tube are fitted tightly onto the main unit and onto the nebulizer.
2. Check whether the nebulizer is empty of medication or overfilled with medication (maximum fill is 6ml)
3. Check whether the nebulizer nozzle is obstructed.
4. Check whether the baffle is inserted properly.

#### Note:

**Never open the device.** There are no serviceable parts within the unit. The compressor requires no lubrication or maintenance.

Distributed by : Arise Medical LLC  
Address : 651 Westminster Road,  
Wilkes-Barre, PA 18702, USA  
Customer Service : +1-866-277-7168  
Website : [www.arisemedical.com](http://www.arisemedical.com)

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Distribuido por : Arise Medical LLC